

## TPPAP100 Anaesthetic Procedure Pack

The TPPAP100 Anaesthetic Procedure Pack is designed to provide all the essential tools and accessories needed to administer a standard anaesthetic. This pack includes 15 high-quality components, each tailored for an efficient and safe procedure.

See Pack Components for a bill of materials.



## Features

- » **Comprehensive Kit:** Each procedure pack includes 15 components tailored to administer a standard anaesthetic.
- » **Sterilised and Ready for Use:** All components are sterilised and double-packed in a sterile environment ready for immediate use.
- » **High Quality:** Made from durable, medical-grade materials to ensure lasting performance and reliability in the operating theatre.
- » **Compliance:** Meets all Health & Safety regulations and is manufactured to ISO 13485:2016 standards.

## Benefits

- » **Enhanced Efficiency:** Having all the necessary tools in one pack streamlines preparation, saving your team time and boosting productivity.
- » **Patient Safety:** A pre-sterilised procedure pack reduces the risk of patient infection.
- » **More Storage Space:** Using one procedure pack significantly improves hospital storage space.
- » **Reduce Waste:** Consolidating all components significantly reduces packaging waste.
- » **Cost Effective:** Combining essential tools into one pack is more economical than purchasing items individually.



Supply Chain  
Product

## Pack Components

Product	Size	Qty
Drape (Fenestrated)	90 x 75 cm	1
Hypodermic Needle (Green)	25G x 38 mm	1
Hypodermic Needle (Yellow)	21G x 16 mm	1
Needle with Filter (Red)	18G x 50 mm	1
Non-woven Swabs	10 X 10 cm	5
Sterilisation Wrap	90 x 90 cm	1
Syringe LS	1 ml	1
Syringe LS	3 ml	1
Syringe LS	5 ml	1
Syringe LS	10 ml	1
Three Compartment Tray	Standard	1



## Instructions of Use

1. Inspect the outer packaging for damage or tampering, check the sterility indicator, and verify the expiration date before opening.
2. Thoroughly wash and sanitise your hands according to standard medical protocols.
3. Open the sterile pack in a clean and sterile environment, following the medical protocol to avoid contamination.
4. Carefully remove the tools and accessories from the pack and arrange them on a sterile surface in the order of use.
5. Ensure all components listed in the pack contents are present and in good condition before beginning the procedure.
6. Use each tool as required by the specific medical procedure, adhering to standard operating protocols and guidelines.
7. Safely dispose items after the procedure.
8. Document the use of the procedure pack and any observations or anomalies during the procedure for future reference.

## Packaging & Storage

**Dimensions per:** Pack: W 32 x L 24 x H 2 (cm)  
 Carton: W: 40 x L 60 x H 45 (cm)

**Weight per:** Pack: 150g  
 Carton: 3kg

**Storage:** Store in a dry environment away from direct sunlight. If stored correctly, each procedure pack has a 3-year shelf life from the date of manufacture.

## Ordering Information

Product Code	NHS NPC	Qty (per carton)
TPPAP100	EVS87301	20

